

## Dexcom CLARITY for Healthcare Professionals

Managing diabetes can be complex. But with Dexcom CLARITY, the newest diabetes reporting and management tool, it's simple and all about patterns!

- Its web-based platform gives you access to glucose data anywhere, anytime - computer or smart device
- You can view, print, email and download PDF reports in minutes

### How does a Healthcare Professional View Data?

Visit [clarity.dexcom.com](http://clarity.dexcom.com) and select "Healthcare Professional"



Home User



Healthcare Professional

There are three ways for healthcare professionals to see data:

1

#### View Data from a Dexcom Receiver

Plug in your patient's Dexcom CGM receiver during an appointment, and you can view or print data. This method is anonymous, no patient name on reports.



2

#### View Shared Data

If your patient has a Dexcom account and the Dexcom CLARITY™ Reports App, they can generate and give you a data-sharing code (see back for details) so you can view their data. Make sure to save your patients sharing code. This method has the patient name on reports.

**NOTE:** Both of the above methods ensure that a user's data is not stored on the computer. The session data cannot be retrieved from the computer, unless a report was saved to the computer as a PDF.

Healthcare professionals can view and print out patient data anytime by entering the patient's sharing code.


3

#### View PDF

Your patient can send you a PDF report or print a report to bring in to the appointment.

## How does a Dexcom User View Data?

Visit [clarity.dexcom.com](https://clarity.dexcom.com) and select "Home User"

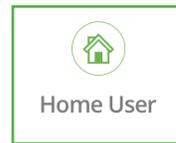

### Sign In or Create Account

Use your existing Dexcom account to sign in to Dexcom CLARITY or create an account.

If you use the Dexcom G5™ Mobile App your data is automatically sent to Dexcom CLARITY. Once you sign in you will be able to see your data automatically.

Follow these steps to upload data from your CGM receiver and see it in Dexcom CLARITY:

### Step 1

#### Download Dexcom CLARITY Uploader

Download the Dexcom CLARITY Uploader, which will connect your CGM receiver with Dexcom CLARITY.

Download

### Step 2

#### Open Downloaded File

When the file has downloaded, open it and follow the installation instructions.

### Step 3

#### Connect Your CGM Receiver

Plug your CGM receiver into your computer and follow the Dexcom CLARITY Uploader instructions.

## How to Share data with a Healthcare Professional?

Download the Dexcom CLARITY App to generate a data-sharing code or view a PDF of your data

### Sign In or Create Account

Use your existing Dexcom account to sign in to the Dexcom CLARITY App or create an account. This only needs to be done the first time using the app.

Generate code and send it to your healthcare professional.

#### How can we help you?

I want to share my data.

Generate Code

I want to view my data.

View a PDF

#### Generate a Sharing Code

I want to share my data for:

3 Months
6 Months
1 Year\*

Generate Code

#### This is Your Sharing Code:

PLRG-MDFF-VPXZ

Please instruct the recipient to enter this code at:

https://clarity.dexcom.com/clinic

Dismiss

\* Recommended so your Healthcare Professional does not need multiple sharing codes